

## HotTooth Keyboard

The HotTooth is a Bluetooth wireless keyboard with full tactile feedback. The keyboard is easy to use and clean. It saves valuable desk space and is flexible so it can be loosely rolled and carried (Carrying pouch included). HotTooth is based on the wireless Bluetooth HID profile. It can be used with PCs, Laptops, Tablet PCs, PDAs and Smartphones with built in Bluetooth HID profile, or via USB Bluetooth bridge with appropriate driver (Bridge with driver sold separately).

### KEYBOARD MUST BE CHARGED AT LEAST TWO HOURS BEFORE 1ST USE

#### Weight:

153 g (0.34 lb.)

#### Dimensions:

330 x114 x 13 mm  
(13.0" x 4.5" x 0.5")

#### Number of keys:

87 keys US layout

#### Power:

Rechargeable Lithium-ion battery.

Charging Cable included.

3 volts (5 volts to charge)

#### System requirements:

Built in Bluetooth HID profile or USB interface for wireless bridge with appropriate driver.

#### Operating Frequency:

2.4 GHz

#### Warranty period:

One-year limited warranty

### DO NOT FOLD OR TIGHTLY ROLL AS THIS MAY DAMAGE KEYBOARD

### Pairing the HotTooth Keyboard:

For Built in Bluetooth HID PCs, Laptops, Tablet PCs and PDAs:

1. Turn on the keyboard by pressing the ON/OFF Button (See the picture on the right). Green light indicates keyboard is on.
2. Press the Pairing Button on the keyboard to enable keyboard pairing mode. Blue light blinks when the keyboard is in pairing mode.
3. Open the Bluetooth manager on your PC, Laptop or PDA and follow the instruction for the software to pair the keyboard.
4. When pairing is complete the blue light stops blinking and it becomes solid green. The keyboard is now ready for use.

Using a USB Bluetooth Bridge: (Bridge with driver sold separately)

1. Turn on the keyboard by pressing the ON/OFF Button (See the picture on the right). Green light indicates keyboard is on.
2. Press the Pairing Button on the keyboard to enable keyboard pairing mode. Blue light blinks when the keyboard is in pairing mode.
3. Connect your USB Bluetooth bridge to the USB port on your PC.
4. Open the Bluetooth manager software for the bridge and follow the instruction for the software to pair the keyboard.
5. When pairing is complete the blue light stops blinking and it becomes solid green. The keyboard is now ready for use.

### Battery status light:

- Red light indicates charging.
- Yellow light indicates low battery power.
- Blinking blue light indicates pairing mode.
- Green light indicates 20% or more battery power.

### Cleaning Instructions for HotTooth Keyboard:

- Clean the keyboard surface by wiping or brushing. Use only rags, sponges or soft bristle brushes.


### DO NOT SUBMERGE THE KEYBOARD DIRECTLY INTO CLEANING FLUID

- The rubber of HotTooth can be cleaned with water, soap, alcohol, 10% bleach solutions, and a variety of chemical disinfectants.

### DO NOT use Petroleum based cleaners, Acetone, Over 10% bleach solutions, Temperature or pressure sterilization, buffing pads or scrubbing pads

### Charging the HotTooth Keyboard:

- Use the Mini USB cable (included) to charge the keyboard. Plug the USB "A" end to your PC and the mini USB "B" end to the USB Charging port connector on the keyboard shown below.
- It takes approximately 2 hours to fully charge the battery.

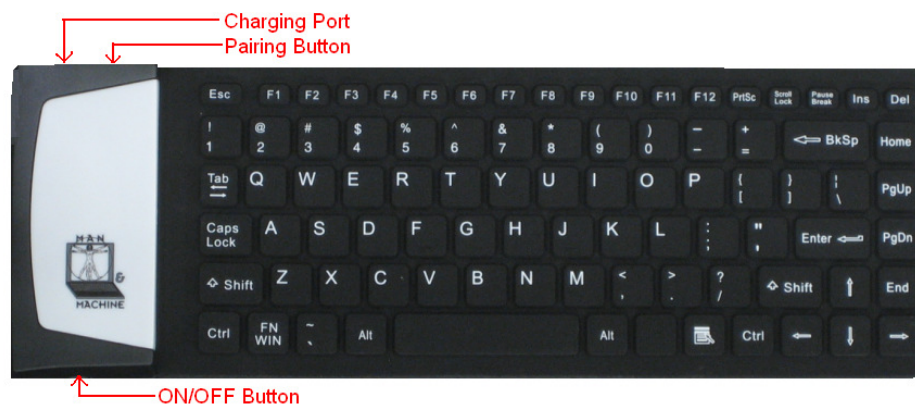

THESE INSTRUCTIONS AND RECOMMENDATIONS HAVE BEEN PREPARED TO ADVISE THE PROPER METHODS FOR CLEANING YOUR PRODUCT. MAN & MACHINE, INC. OR THE LOCAL DISTRIBUTOR CANNOT ASSUME ANY RESPONSIBILITY FOR THE APPLICATION OF THEM. IF THERE ARE ANY QUESTIONS, PLEASE CONTACT PLACE OF PURCHASE OR MMI PRIOR TO CLEANING. MAN & MACHINE DOES NOT CONSIDER THE HOTTOOTH KEYBOARD TO BE A MEDICAL KEYBOARD FOR INFECTION CONTROL PURPOSES DUE TO THE INABILITY TO MECHANICALLY CLEAN ALL SURFACES AND AREAS ON THE KEYBOARD. ALL SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE.

You can email us at: [info@mmimd.com](mailto:info@mmimd.com)  
or call us at: (301) 341-4900  
In Europe, contact +31-23-5450185  
[www.man-machine.com](http://www.man-machine.com)  
Copyright Man & Machine, Inc. 2008

