

Really Cool Keyboard

The Really Cool is a waterproof and contaminant proof keyboard with tactile feedback keys. The sealed and rigid surface is ideal for industrial, marine, and medical use, as well as other applications where moisture, cleaning, and durability are a consideration. And, because the Really Cool is also quieter than "standard" keyboards, they are great for applications where noise generated from "key-clicking" is a problem. This keyboard can also be equipped with our Backlight option where LED's are installed under each key for use in dark or low light conditions, and our "MAGFIX" option where magnets are installed in the base to help hold it in place on ferrous surfaces.

Weight:

817 g (1.8 lb.)

Dimensions:

381 mm x 140 mm x 15.3 mm
15" x 5.5" x 0.6"

Cable length:

1.83 m (6')

Storage temperature:

-20 °C - 70 °C
-4 °F - 158 °F

Operating temperature:

0 °C - 70 °C
32 °F - 158 °F

Voltage:

5 Volt

Current input:

26 mA - (w/o Backlight)
Up to max 300 mA (with Backlight)

Composition:

100% Silicone Rubber
PVC Cable
100% Latex free
100% RoHS Compliant

System requirements:

USB interface
(Not PS/2 Compatible)

Number of keys:

104 = US Layout with Windows™
plus additional function keys

Warranty period:

Limited lifetime warranty

*Specifications subject to change
without notice*


Installing Your Really Cool Keyboard

Attach the keyboard cable to a USB computer port.
(You do not have to turn off your computer. There is no software to install.)

You can email us at: info@mmimd.com
or call us at: (301) 341-4900
In Europe, contact +31-23-5450185
www.man-machine.com



For Keyboards with Backlight Option Function:

- Press the key on the top row with the  symbol to turn on, adjust, and turn off the backlighting feature.
- Pressing this key once turns on the backlight with 30% light intensity.
- A second time turns on the backlight with a 60% light intensity.
- A third time turns on the backlight at 100% intensity.
- A fourth time turns the backlight off.

Cleaning Instructions for Waterproof Keyboards and Mice:

- To clean the Really Cool Keyboard while it is plugged into the PC, press the left and right Control ("Ctrl") keys together for 3 seconds to lock the keyboard. After cleaning, press both keys again for 3 seconds to unlock keyboard.
- Clean the surface of the keyboard by wiping, brushing, spraying or dipping it. Use only rags, sponges or soft bristle brushes.
DO NOT place the USB connector into the fluid.
- Either wipe with dry cloth or allow to air dry.

Disinfectants recommended by Man & Machine, Inc.:

- **Chlorine based (max 10% solution)** - Chlorox, Novalsan, etc.
- **Formaldehyde based** - Formaldehyde Solution 37%, Vinco, Formaldegen, etc.
- **Glutaraldehyde based** - Aldacide 200, Lysofume, Wavicide, etc.
- **Phenol based** - Lysol I.C., Beaucoup, Magna Clean, Tek-Trol, etc.
- **Alcohol based** - Cavicide, Isopropyl Alcohol, Medicide, Dürr FD 322, FD 333 and FD 350, etc.
- **Oxidizing** - Hydrogen Peroxide 3% Solution, Lifeguard Series, etc.
- **Quaternary Ammonium** - Roccal-D, Bacto-Sep, Parvosol, Dürr FD 300 2%, Dürr FD 312 2%, etc.
- **Soap and water**

DO NOT USE:

Petroleum based cleaning fluids, such as Gasoline, Acetone, over 10% bleach solutions, temperature or pressure sterilization, buffing pads or scrubbing pads.

THESE INSTRUCTIONS AND RECOMMENDATIONS HAVE BEEN PREPARED TO ADVISE THE PROPER METHODS FOR CLEANING YOUR PRODUCT. MAN & MACHINE, INC. OR THE LOCAL DISTRIBUTOR CANNOT ASSUME ANY RESPONSIBILITY FOR THE APPLICATION OF THEM. IF THERE ARE ANY QUESTIONS, PLEASE CONTACT PLACE OF PURCHASE OR MMI PRIOR TO CLEANING.



Copyright Man & Machine, Inc. 2009

PRODUCT WARRANTY STATEMENT

Limited Lifetime Warranty

Really Cool/Really Cool +, Slim Cool/Slim Cool + Keyboards and Mighty Mouse 5/Petite Mouse

One Year Warranty

U Cool, U Hot, Simply Cool, Cool4Kids, Hot Tooth, CoolMir Keyboards

Man & Machine, Inc. (“Manufacturer”) warrants to the original purchaser (the “Customer”) that the products it manufactures (“Product”) shall be free from defects in material and workmanship at time of shipment. Manufacturer’s obligations under this Warranty shall be limited to repairing or replacing with the same or comparable item, at its option, free of charge, any product returned to Manufacturer freight prepaid. Manufacturer shall have no obligation under this Warranty or otherwise if (1) the Product has been damaged by abuse, negligence, accident, mishandling, lightning or flood, or other Acts of God, (2) the Product has been affected by normal wear and tear, (3) the Product has not been operated in accordance with its operating instructions, (4) the Product has been altered or repaired by anyone outside Manufacturer (5) adaptations or accessories have been made or attached to the Product which, in Manufacturer’s sole determination, have adversely affected its performance, safety or reliability.

If a Product should malfunction or fail during the warranty period stated above, contact the original place of purchase for return options. If purchased directly from Man & Machine, Inc., please submit a Return Material Authorization (RMA). This form can be located on our website at <http://www.man-machine.com/replacement.php>. Please reference your original order number, original order information and complete description of the problem on the form. Returned Products must include the RMA number clearly marked on outside of the package. Manufacturer will not be responsible for any unnecessary items included with any returned Product.

THIS WARRANTY IS THE EXCLUSIVE WARRANTY FOR ANY PRODUCT. MANUFACTURER SPECIFICALLY DISCLAIMS ANY AND ALL OTHER WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING ANY WARRANTIES OF FITNESS FOR A PARTICULAR PURPOSE OR OF MERCHANTABILITY. MANUFACTURER SHALL NOT BE LIABLE IN TORT, INCLUDING NEGLIGENCE OR STRICT LIABILITY, AND SHALL HAVE NO LIABILITY AT ALL FOR INJURY TO PERSONS OR PROPERTY. MANUFACTURER’S LIABILITY FOR FAILURE TO FULFILL ITS OBLIGATION UNDER THIS LIMITED WARRANTY OR ANY OTHER LIABILITY IN CONNECTION WITH A PRODUCT SHALL BE LIMITED TO THE AMOUNT OF THE PURCHASE PRICE RECEIVED BY MANUFACTURER FOR THE PRODUCT. THE REMEDIES STATED IN THIS LIMITED WARRANTY ARE THE CUSTOMER’S EXCLUSIVE REMEDIES AGAINST MANUFACTURER REGARDING ANY PRODUCT. UNDER NO CIRCUMSTANCES SHALL MANUFACTURER BE LIABLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES, INCLUDING LOST PROFITS AND REVENUES, INCONVENIENCE OR INTERRUPTIONS IN OPERATIONS, OR ANY OTHER COMMERCIAL OR ECONOMIC LOSSES OF ANY KIND. THESE LIMITATIONS AND DISCLAIMERS ARE NOT MADE BY MANUFACTURER WHERE PROHIBITED BY LAW. SOME STATES PROHIBIT LIMITATIONS OF WARRANTIES AND THE CUSTOMER MAY HAVE ADDITIONAL RIGHTS IN THOSE STATES.

All items returned for warranty repair/replacement must be sent freight prepaid and insured to either the original place of purchase or to the Manufacturer:

Man & Machine, Inc.
3706 West Street
Landover, MD 20785

All warranty repairs or replacements will be returned freight prepaid by ground service to any destination within the Continental United States of America. Charges for repairs/replacements shipped back outside this region are the responsibility of the end user.